


## SHADE COMBINATION CHART

		A1	A2	A3	A3.5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4	A0	B0		
OPAQUE	FOUNDATION OPAQUE											FO						FO			
	MARGIN OPAQUE											MO						—			
	OPAQUE ORIGINAL SHADE	OA1	OA2	OA3	OA3.5	OA4	OB1	OB2	OB3	OB4	OC1	OC2	OC3	OC4	OD2	OD3	OD4	OW			
	OPAQUE CLASSICAL SHADE	OA1-C	OA2-C	OA3-C	OA3.5-C	OA4-C	OB1	OB2	OB3	OB4	OC1	OC2	OC3	OC4	OD2	OD3	OD4	OW			
CERVICAL	OPAQUS DENTIN, OPAQUS DENTIN INTENSIVE*1 ORIGINAL SHADE	ODA2	ODA3	ODA3.5	ODA4	ODI3	ODB1	ODB2	ODB3	ODB4	ODI2	ODC1	ODC2	ODC3	ODC4	ODI5	ODD3	ODD4	ODI6	ODA1	ODB1
	OPAQUS DENTIN, OPAQUS DENTIN INTENSIVE*1 CLASSICAL SHADE	ODA2-C	ODA3-C	ODA3.5-C	ODA4-C		ODB1	ODB2	ODB3	ODB4	ODI2	ODC1	ODC2	ODC3	ODC4	ODI5	ODD3	ODD4	ODI6	ODA1	ODB1
DENTIN	OPAQUS DENTIN ORIGINAL SHADE	ODA1	ODA2	ODA3	ODA3.5	ODA4	ODB1	ODB2	ODB3	ODB4	ODC1	ODC2	ODC3	ODC4	ODD2	ODD3	ODD4	ODA0	ODB0		
	OPAQUS DENTIN CLASSICAL SHADE	ODA1-C	ODA2-C	ODA3-C	ODA3.5-C	ODA4-C	ODB1	ODB2	ODB3	ODB4	ODC1	ODC2	ODC3	ODC4	ODD2	ODD3	ODD4	ODA0	ODB0		
	DENTIN ORIGINAL SHADE	DA1	DA2	DA3	DA3.5	DA4	DB1	DB2	DB3	DB4	DC1	DC2	DC3	DC4	DD2	DD3	DD4	DA0	DB0		
	DENTIN CLASSICAL SHADE	DA1-C	DA2-C	DA3-C	DA3.5-C	DA4-C	DB1	DB2	DB3	DB4	DC1	DC2	DC3	DC4	DD2	DD3	DD4	DA0	DB0		
ENAMEL	ENAMEL	E2	E3	E4	E1	E2	E3	E2	E3	E4	E2	E3	E4	E2	E3			E0			
	ENAMEL INTENSIVE											E11						E0			
TRANSLUCENT	CERVICAL TRANSLUCENT	*2	CT2	CT4	CT3	*2	CT2	CT4	*2	CT2	CT3	CT2	CT4					—			

\*1: OPAQUS DENTIN and OPAQUS DENTIN INTENSIVE are used as cervical colors.

\*2: The use of ENAMEL INTENSIVE (E11) will give almost the same effects as CERVICAL TRANSLUCENT.

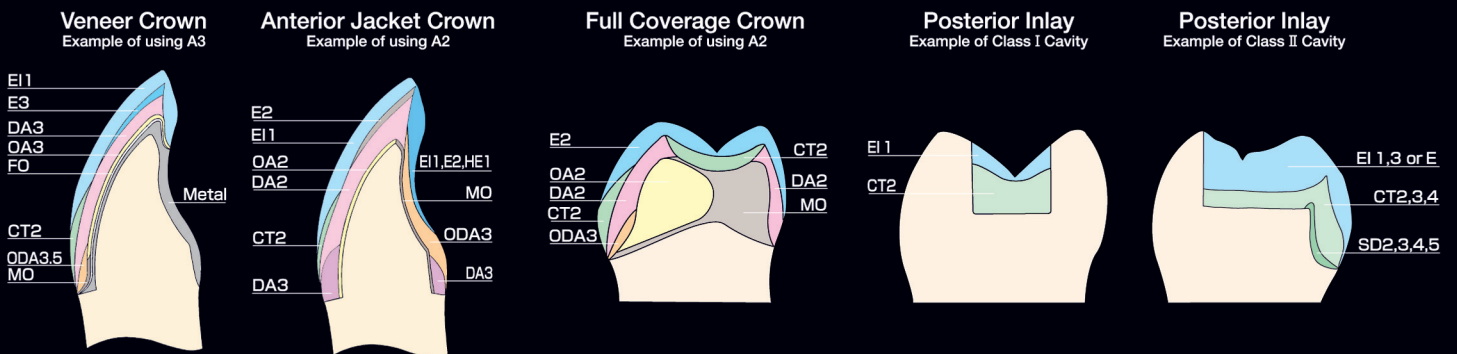

# GRADIA CHARACTERIZATION

SHOULDER DENTIN															
	SD2	SD3	SD4	SD5	SD7	SD8									
OPAQUE DENTIN INTENSIVE															
	OD1	OD2	OD3	OD4	OD5	OD6									
ENAMEL INTENSIVE															
	EI0	EI1	EI3	EI5											
HALO ENAMEL															
	HE1														
PEARL ENAMEL															
	PE1	PE3													
TRANSLUCENT															
	T0	T1	T2	T4	T5										
CERVICAL TRANSLUCENT															
	CT2	CT3	CT4												
MAMELON STAIN															
	MS2	MS3	MS5												
INTENSIVE COLOR															
	IC0	IC1	IC2	IC3	IC4	IC5	IC6	IC7	IC8	IC9	IC10	IC11	IC12	IC13	IC14
FLOWABLE COMPOSITE															
	F-WT	F-WE													

## IRRADIATION TIME CHART

	LABOLIGHT LV-III. II		STEPLIGHT SL-I
	Pre-cure	Final Cure	Pre-cure
FOUNDATION OPAQUE OPAQUE MARGIN OPAQUE GUM OPAQUE GUM OPAQUE MODIFIER	1 min.	—	—
DENTIN OPAQUE DENTIN OPAQUE DENTIN INTENSIVE SHOULDER DENTIN ENAMEL ENAMEL INTENSIVE HALO ENAMEL PEARL ENAMEL TRANSLUCENT CERVICAL TRANSLUCENT GUM GUM MODIFIER GUM TRANSLUCENT	30 sec.	3 min.	10 sec.*
INTENSIVE COLOR MAMELON STAIN FLOWABLE COMPOSITE	30 sec.	—	10 sec.*


\*For one surface of a single crown

## GRADIA GUM SHADE COMBINATION CHART


OPAQUE	GUM OPAQUE							
	GUM OPAQUE MODIFIER							
BODY (Paste)	GUM							
MODIFIER (Gel)	GUM MODIFIER							
TRANSLUCENT (Gel)	GUM TRANSLUCENT							

GUM FIBER	
	GF71

Example of Build-up

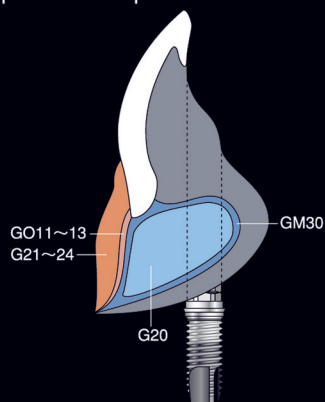

Example of Multiple-Layering Technique

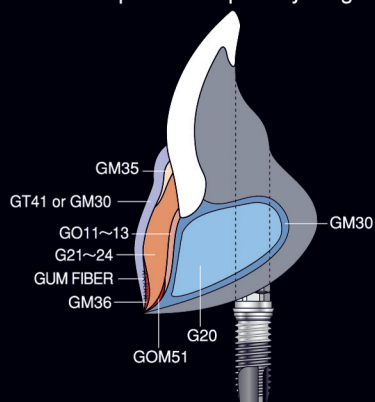

GC EUROPE N.V.  
Head Office  
Interleuvenlaan 13  
B - 3001 Leuven  
Tel. +32.16.39.80.50  
Fax. +32.16.40.02.14  
info@gceurope.com  
www.gceurope.com

**GC**  
FIRST IS QUALITY