Prior to use, carefully read the instructions for use.

## ORTHO CONDITIONER

ORTHODONTIC CLEANING AGENT

For use only by a dental professional in the recommended indications.

#### RECOMMENDED INDICATIONS

ORTHO CONDITIONER is a 10% polyacrylic acid solution designed to mildly etch and condition enamel, thus enhancing the bondability of glass ionomer to the tooth surface.

#### CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

#### DIRECTIONS FOR USE

- Clean tooth surface with pumice and water, prophy cup or brush. Rinse thoroughly and dry.
- Dispense a drop of ORTHO CONDITIONER into a dappen dish or similar container.

Note :

After dispensing, remove any material remaining at the tip of the nozzle using wet gauze, etc. Close the bottle immediately after use.
- Using a cotton pellet, apply ORTHO CONDITIONER for 20 seconds.
- Rinse ORTHO CONDITIONER from the tooth. Keep the surface moist. The conditioned tooth is now ready for the application of GC Fuji ORTHO or GC Fuji ORTHO LC glass ionomer cement.

#### STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F). (Shelf life: 3 years from date of manufacture)

#### PACKAGE

Bottle of 25g (23.8mL)

#### CAUTION

- In case of contact with oral tissue or skin, flush immediately with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.

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