**GC Fuji TRIAGE®**

RADIOPAQUE GLASS IONOMER PROTECTION AND TEMPORARY RESTORATIVE MATERIAL

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

1. Fissure protection.
2. Root surface protection.
3. Hypersensitivity prevention and control.
4. Protection for immature enamel.
5. Temporary filling of endodontic access.
6. Intermediate restorations.

CONTRAINDICATIONS

1. Pulp capping.
2. Avoid use of this product in patients with known glass ionomer cement allergies.

DIRECTIONS FOR USE

Pink Shade is command set, WHITE Shade is chemical set only.

Powder / Liquid Ratio (g / g)	1.8 / 1.0
Mixing Time (sec.)	20-25*
Working Time (min., sec.)	1' 40"
Net Setting Time (min., sec.)	2' 30"
Final Finishing Commencing Time	6'00"
Final Finishing Commencing Time if light cured (PINK Command set)	4'00"
Test conditions : Temperature (23+/-1°C), Relative humidity (50+/-5%)	

Testbedingungen : Temperatur (23+/-1°C),
Relative Feuchte (50+/-5%)**A. FISSURE PROTECTION****ROOT SURFACE PROTECTION****HYPERSENSITIVITY PREVENTION AND CONTROL****PROTECTION FOR IMMATURE ENAMEL****PROTECTION OF THE TOOTH SURFACES (e.g. for fissure protection or root surface protection)**

- a) After cleaning the tooth surfaces (prophylaxis with pumice and water) in usual manner, rinse thoroughly with water. Avoid aggravating the operculum.

Note: If extra retention is desired, application of CAVITY CONDITIONER (10 seconds) or DENTIN CONDITIONER (20 seconds) is recommended. The conditioner should be rinsed off thoroughly after application.

- b) Dry by blotting with a cotton pellet or gently blowing with an air syringe. Avoid DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening).

2. Powder and Liquid Dispensing

- a) The standard powder to liquid ratio is 1.8g / 1.0g. (1 level scoop of powder to 1 drop of liquid).
- b) For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- c) Make sure the liquid nozzle is clean and dry before dispensing liquid. Turn the liquid bottle horizontal and hold in this position while dispensing. Then invert and hold the liquid bottle vertically and squeeze gently to dispense a bubble free drop of liquid. After dispensing, wipe any residual liquid off the nozzle.
- d) Close both caps tightly immediately after use.

3. Mixing

Dispense the required amounts of powder and liquid onto the pad. Use the plastic spatula, divide the powder into 2 equal parts. Spread the liquid over the pad (Fig. A-2) and mix the first portion with all the liquid for 10 seconds. Incorporate the remaining portion and mix the whole amount thoroughly for 10-15 seconds (total time within 25 seconds).

Working time is 1 minutes 40 seconds from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.

4. Placement

- a) Mix the mixed material using a suitable placement instrument or brush and apply to the tooth surface (Fig. A-3). Then use a brush to spread a thin film of GC Fuji TRIAGE® directly over the root surface or hypersensitive area or over the occlusal surface and into the pits and fissures.
- Note : If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.
- b) After placement, when the material starts to lose the glossy appearance (or after curing with the light curing device), apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) to the sealed area and the margins using a cotton pellet or sponge (Fig. A-4).
- c) Finishing under air water spray can be performed 6 minutes from start of cure (chemically set) or 4 minutes if light cured. Use a superfine diamond bur or a silicone finishing point (Fig. A-5).
- d) Apply GC Fuji VARNISH or GC Fuji COAT LC to the area again (Fig. A-6).

B. TEMPORARY FILLING OF ENDODONTIC ACCESS**1. Cleaning the pulp chamber**

- a) After appropriate pulp treatment, clean and gently dry the pulp chamber and all areas (Fig. B-1).
- b) Fill the chamber with a cotton pellet (Fig. B-2).

2. Powder / Liquid Dispensing and Mixing

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CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. DO NOT mix powder or liquid with any other glass ionomer components.

4. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

5. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gc-europe.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device for any visible damage. Discard device if damaged. DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

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Note:

If extra retention is desired, application of CAVITY CONDITIONER (10

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